

INSTRUCTIONS FOR USE

 **MOUNJARO™**

tirzepatide injection

for subcutaneous use



2.5 mg/0.5 mL single-dose pen

5 mg/0.5 mL single-dose pen

7.5 mg/0.5 mL single-dose pen

10 mg/0.5 mL single-dose pen

12.5 mg/0.5 mL single-dose pen

15 mg/0.5 mL single-dose pen

Once-weekly

Lilly


Important information you need to know before injecting MOUNJARO

Read this “Instructions for Use” and the Patient Information before using your MOUNJARO Pen and each time you get a refill. There may be new information. This information does not take the place of talking to your healthcare provider about your medical condition or treatment.

Talk to your healthcare provider about how to inject MOUNJARO the right way.

- MOUNJARO is a single-dose prefilled pen.
- MOUNJARO is used 1 time each week.
- Inject under the skin (subcutaneously) only.
- You or another person can inject into your stomach (abdomen) or thigh.
- Another person should give you the injection in the back of your upper arm.

Guide to parts of MOUNJARO Pen

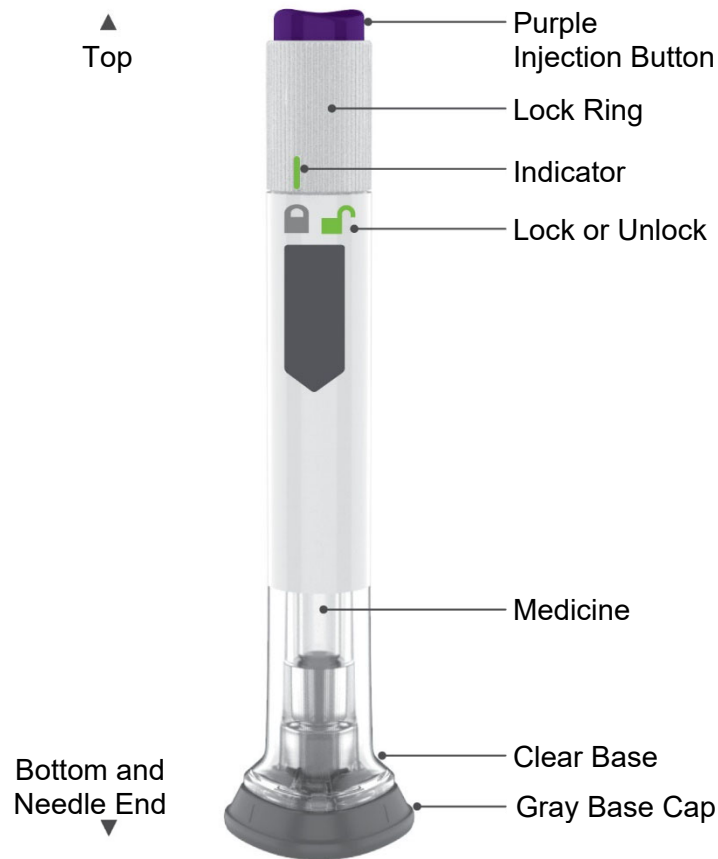

Preparing to inject MOUNJARO

Remove the Pen from the refrigerator.

Leave the gray base cap on until you are ready to inject.

Check the Pen label to make sure you have the right medicine and dose, and that it has not expired.



Inspect the Pen to make sure that it is not damaged.


Make sure the medicine is colourless to slightly yellow.

Do not use the medicine if it is: frozen, cloudy, or has particles.

Wash your hands.


Choose your injection site

Your healthcare provider can help you choose the injection site that is best for you. Change (rotate) your injection site each week. You may use the same area of your body, but be sure to choose a different injection site in that area.

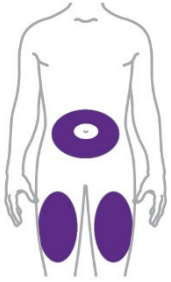

You or another person can inject the medicine in your stomach (abdomen) or thigh,

OR

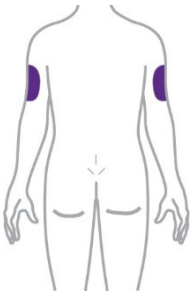


Another person should give you the injection in the back of your upper arm.

Step

1

Pull off the gray base cap

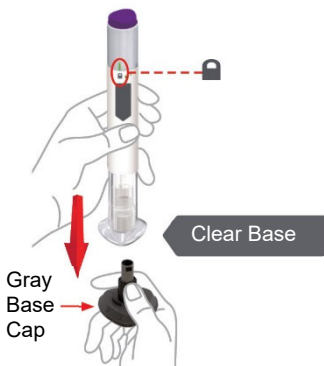

Make sure the Pen is **Locked**.

Do not unlock the Pen until you place the clear base on your skin and are ready to inject.

Pull the gray base cap straight off and throw it away in your household trash.

Do not put the gray base cap back on – this could damage the needle.

Do not touch the needle.


Step

2

Place Clear Base on skin, then Unlock



Place the clear base flat against your skin at the injection site. The Purple button should be away from the skin.

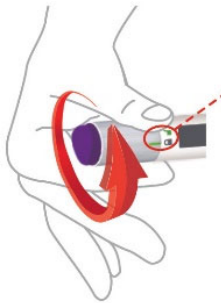

Unlock by turning the lock ring toward the unlock icon.

Step

3

Press and Hold up to 10 seconds



Press and Hold the purple injection button.

Listen for two loud clicks:

- First click = injection started
- Second click = injection completed



You will know your injection is complete when the gray plunger is visible.

After your injection, place the used Pen in a sharps container.

Disposing of your used Pen

- Put your used Pen in a sharps disposal container right away after use. **Do not** throw away (dispose of) Pens in your household trash.
- If you do not have a sharps disposal container, you may use a household container that is:
 - made of a heavy-duty plastic,
 - can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
 - upright and stable during use,
 - leak-resistant, and



- properly labeled to warn of hazardous waste inside the container.
- When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be local laws about how you should throw away used needles and syringes.
- **Do not** recycle your used sharps disposal container.
- Keep the disposal container out of the sight and reach of children.

Storage and handling

- Store your Pen in the refrigerator between 2°C to 8°C.
- You may store your Pen at room temperature below 30°C for up to a total of 21 days.
- **Do not** freeze your Pen. If the Pen has been frozen, throw the Pen away and use a new Pen.
- Store your Pen in the original carton to protect your Pen from light.
- The Pen has glass parts. Handle it carefully. If you drop the Pen on a hard surface, **do not** use it. Use a new Pen for your injection.
- Keep your MOUNJARO Pen and all medicines out of the sight and reach of children.

Commonly asked questions

What if I see air bubbles in my Pen?

Air bubbles are normal.

What if my Pen is not at room temperature?

It is not necessary to warm the Pen to room temperature.

What if I unlock the Pen and press the purple injection button before pulling off the gray base cap?

Do not remove the gray base cap. Throw away the Pen and get a new Pen.

What if there is a drop of liquid on the tip of the needle when I remove the gray base cap?

A drop of liquid on the tip of the needle is normal. **Do not** touch the needle.


Do I need to hold the injection button down until the injection is complete?

This is not necessary, but it may help you keep the Pen steady against your skin.

I heard more than 2 clicks during my injection—2 loud clicks and 1 soft one. Did I get my complete injection?

Some people may hear a soft click right before the second loud click. That is the normal operation of the Pen. **Do not** remove the Pen from your skin until you hear the second loud click.

I am not sure if my Pen worked the right way.

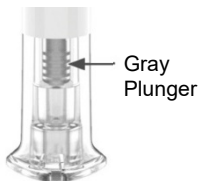

Check to see if you have received your dose. Your dose was delivered the right way if the gray plunger is visible. Also, see **Step 3** of the instructions.

If you do not see the gray plunger, contact Lilly at 1-888-545-5972 for further instructions. Until then, store your Pen safely to avoid an accidental needle stick.

What if there is a drop of liquid or blood on my skin after my injection?

This is normal. Press a cotton ball or gauze over the injection site. **Do not** rub the injection site.

Other Information

- If you have vision problems, **do not** use your Pen without help from a person trained to use the MOUNJARO Pen.

Where to learn more

- If you have questions or problems with your MOUNJARO Pen, contact Lilly at 1-888-545-5972 or call your healthcare provider.
- For more information about MOUNJARO Pen, visit our website at www.lilly.ca.

Manufactured by:
Eli Lilly Canada Inc.
P.O. Box 73
Toronto, Ontario
M5X 1B1

MOUNJARO is a trademark owned by or licensed to Eli Lilly and Company, its subsidiaries or affiliates.

Copyright © 2022, Eli Lilly and Company. All rights reserved.

Approval Date: NOV 23, 2022

MOU-0001-IFU-20221123