


INSTRUCTIONS FOR USE
EBGLYSS™ [EHB-glihs]
(lebrikizumab-lbkz)
injection, for subcutaneous use
Single-Dose Prefilled Syringe with Needle Shield

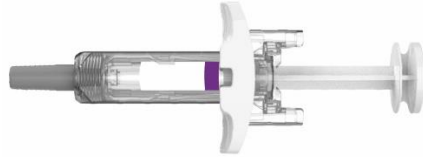

This Instructions for Use contains information on how to inject EBGLYSS.

Before you use the EBGLYSS Prefilled Syringe with Needle Shield (Prefilled Syringe), read and carefully follow all the step-by-step instructions.

Important information you need to know before injecting EBGLYSS


- Your healthcare provider should show you how to prepare and inject EBGLYSS using the Prefilled Syringe. **Do not** inject yourself or someone else until you have been shown how to inject EBGLYSS.
- Keep these Instructions for Use and read it as needed.
- Each EBGLYSS Prefilled Syringe contains 1 dose of EBGLYSS. **The Prefilled Syringe is for one-time use only.**
- The EBGLYSS Prefilled Syringe contains glass parts. Handle it carefully. If you drop it on a hard surface, **do not** use it. Use a new EBGLYSS Prefilled Syringe for your injection.
- Your healthcare provider may help you decide where on your body to inject your dose. You can also read the **Choose and clean your injection site** section of these instructions to help you choose which area can work best for you.
- If you have vision problems, **do not** use the EBGLYSS Prefilled Syringe without help from a caregiver.
- See **Storing EBGLYSS** for important storage information.

INSTRUCTIONS FOR USE


Before you use the EBGLYSS Prefilled Syringe, read and carefully follow all the step-by-step instructions.

Parts of the EBGLYSS Prefilled Syringe with Needle Shield

Before Use

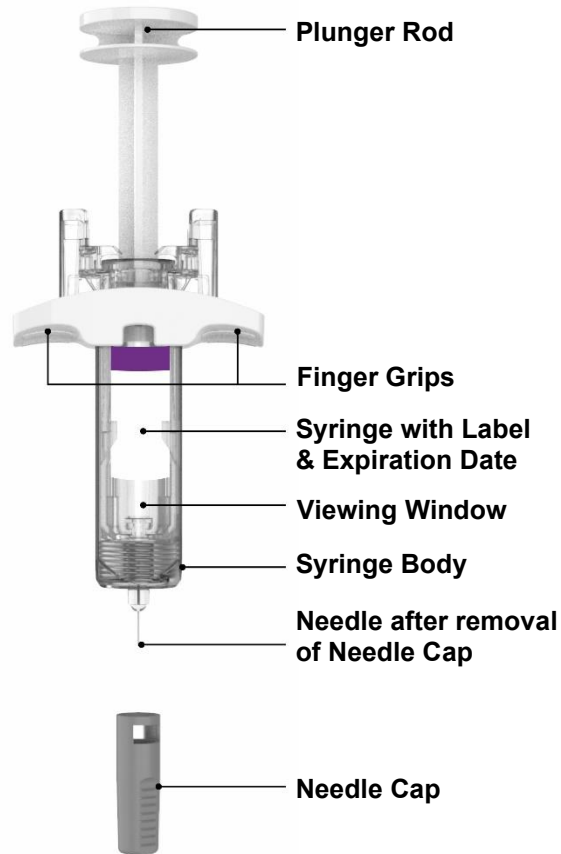

After Use

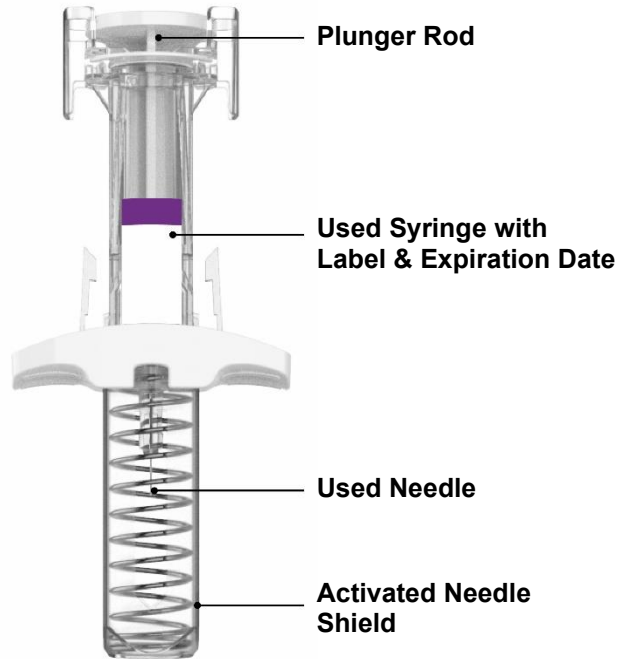


Preparing to inject EBGLYSS

Gather Supplies and EBGLYSS Prefilled Syringe

- EBGLYSS Prefilled Syringe from the refrigerator
- 1 alcohol wipe
- 1 cotton ball or piece of gauze
- 1 sharps disposal container (see **Disposing of EBGLYSS**)

Remove Prefilled Syringe from carton

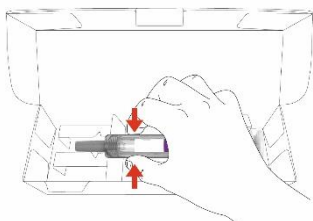

Remove the EBGLYSS Prefilled Syringe from the carton by holding the middle of the Syringe Body and then place the EBGLYSS Prefilled Syringe on a clean flat surface.

Leave the needle cap on the syringe until you are ready to inject.

Do not pull back on the Plunger Rod at any time.


Wait 45 minutes

With the Needle Cap on, allow the Prefilled Syringe to warm up to room temperature for 45 minutes before injecting.

- **Do not** warm up the EBGLYSS Prefilled Syringe with a microwave, or hot water, or direct sunlight.
- **Do not** use the Prefilled Syringe if the medicine is frozen.

Inspect the Prefilled Syringe and the medicine

Make sure you have the right medicine. The medicine inside should be clear. It may be colorless to slightly yellow to slightly brown. Some air bubbles are normal.

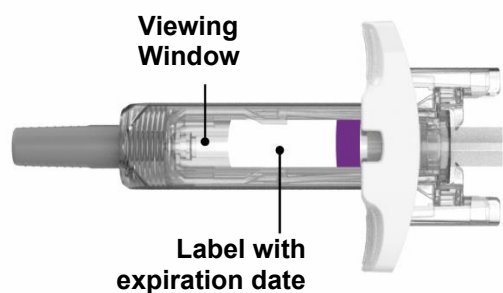

Note: You may gently rotate the plunger rod to view the syringe label if needed.


Do not use the Prefilled Syringe (see **Disposing of EBGLYSS**) if the:

- Needle Cap is missing or not securely attached
- Prefilled Syringe looks damaged
- medicine is cloudy, is discolored, or has particles
- expiration date printed on the label has passed.


Wash your hands with soap and water

Choose and clean your injection site

Your healthcare provider can help you choose the injection site that is best for you. **Clean the injection site with an alcohol wipe and let dry.**

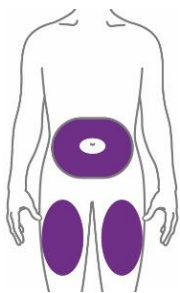

You or another person may inject into these areas.

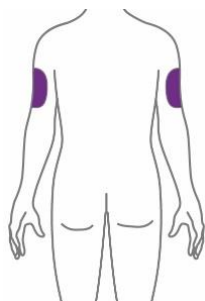

Another person should inject into this area.

- **Stomach area (abdomen) –**
 - At least 2 inches away from the belly button (navel).
- **Front of thigh –**
 - At least 2 inches above the knee and 2 inches below the groin.
- **Back of upper arm –**
 - Another person should inject into the back of your upper arm.

Do not inject in the exact same spot every time.

Do not inject into areas where the skin is tender, bruised, red, hard, or in an area of the skin that is affected by eczema (atopic dermatitis) or other skin lesions.

Injecting EBGLYSS


1 **Remove Needle Cap**

Hold the EBGLYSS Prefilled Syringe in the middle of the Syringe Body and pull the Needle Cap straight off.

Do not put the Needle Cap back on; this could damage the needle.

Do not touch the Needle.


2 **Pinch the injection site and insert the Needle**

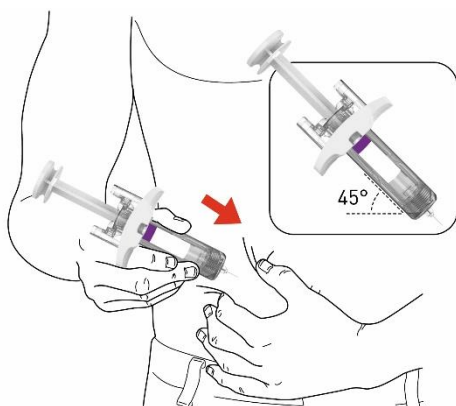

Gently pinch a fold of skin at the injection site with one hand.

Insert the Needle completely into the fold of skin at about a 45° angle with your other hand.

3

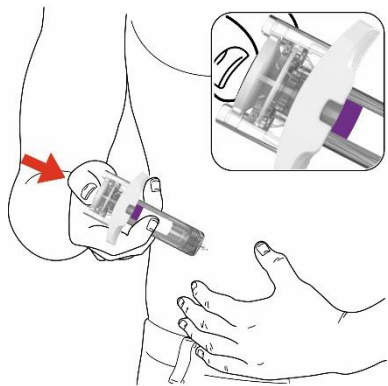
Push in the Plunger Rod

Gently relax the pinch while keeping the needle in place. Push the Plunger Rod all the way down as far as it will go to inject all of the medicine.

Note: It is normal to feel some resistance.

You have to give the full dose to activate the Needle Shield.

4

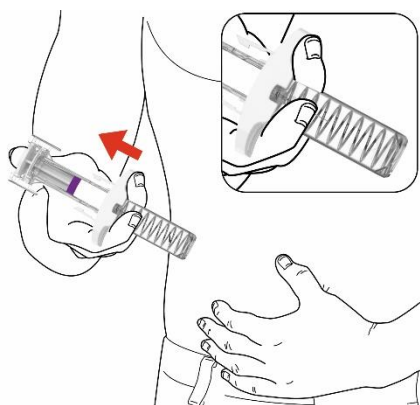Release the Plunger Rod


Lift your thumb to release the Plunger Rod until the Needle is covered by the Needle Shield.

Lightly press a cotton ball or gauze on the injection site if you see any blood.

Do not rub your skin after the injection.

Disposing of EBGLYSS

Dispose of (throw away) the used Prefilled Syringe

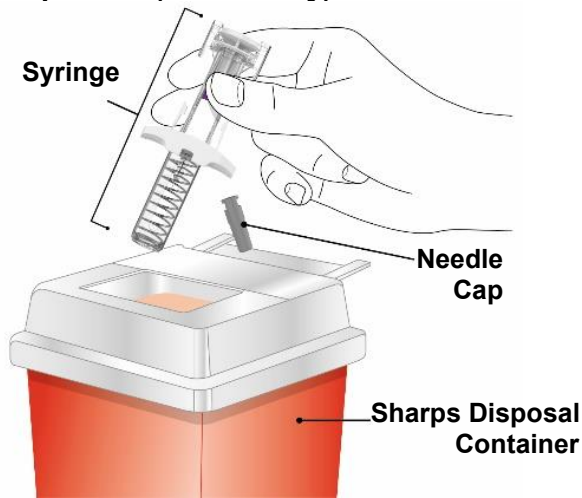

Put the used EBGLYSS Prefilled Syringe and Needle Cap in an FDA-cleared sharps disposal container right away after use.

Do not throw away (dispose of) the EBGLYSS Prefilled Syringe in your household trash.

If you do not have an FDA-cleared sharps disposal container, you may use a household container that is:

- made of a heavy-duty plastic,
- able to be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
- upright and stable during use,
- leak-resistant, and
- properly labeled to warn of hazardous waste inside the container.

When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container.

There may be local laws about how you should throw away needles and syringes.

For more information about safe sharps disposal, and for specific information about sharps disposal in the state you live in, go to the FDA website at: <http://www.fda.gov/safesharpsdisposal>

Do not recycle your used sharps disposal container.

Storing EBGLYSS

- Store your Prefilled Syringe in a refrigerator between 36°F to 46°F (2°C to 8°C).
- EBGLYSS can be stored at room temperature up to 7 days in the original carton. **Do not** store above 86°F (30°C). Throw away (dispose of) EBGLYSS that has been left at room temperature for longer than 7 days.
- Store your Prefilled Syringe in the original carton to protect from light until use.
- **Do not** freeze your Prefilled Syringe. **Do not** shake your Prefilled Syringe.
- **Do not** microwave your Prefilled Syringe, or run hot water over it, or leave it in direct sunlight.
- Throw away (dispose of) your Prefilled Syringe if any of the above conditions are not followed.

Keep your Prefilled Syringes and all medicines out of the reach of children.

If you have more questions about how to use the EBGLYSS Prefilled Syringe:

- Call your healthcare provider
- Call 1-800-Lilly-Rx (1-800-545-5979)
- Visit www.ebglyss.com



**Scan this code to launch
www.ebglyss.com**

Read the Patient Information insert for EBGLYSS inside this box to learn more about your medicine.

Eli Lilly and Company
Indianapolis, IN 46285, USA
US License Number 1891

EBGLYSS is a trademark of Eli Lilly and Company.

Copyright © 2024, Eli Lilly and Company. All rights reserved.

This Instructions for Use has been approved by the U.S. Food and Drug Administration.

Issued: September 2024

EBG-0001-PFS-IFU-202409