

**INSTRUCTIONS FOR USE**  
**ZEPBOUND™ (ZEHP-bownd)**  
**(tirzepatide)**  
**injection, for subcutaneous use**



**2.5 mg/0.5 mL single-dose pen**  
**5 mg/0.5 mL single-dose pen**  
**7.5 mg/0.5 mL single-dose pen**  
**10 mg/0.5 mL single-dose pen**  
**12.5 mg/0.5 mL single-dose pen**  
**15 mg/0.5 mL single-dose pen**  
**use 1 time each week**


**Important information you need to know before injecting ZEPBOUND**

**Read this Instructions for Use and the Medication Guide before using your ZEPBOUND pen and each time you get a refill.** There may be new information. This information does not take the place of talking to your healthcare provider about your medical condition or treatment.

Talk to your healthcare provider about how to inject ZEPBOUND the right way.

- ZEPBOUND is a single-dose prefilled pen.
- ZEPBOUND is used 1 time each week.
- Inject under the skin (subcutaneously) only.
- You or another person can inject into your stomach (abdomen) or thigh.
- Another person can inject into the back of your upper arm.

## Guide to parts

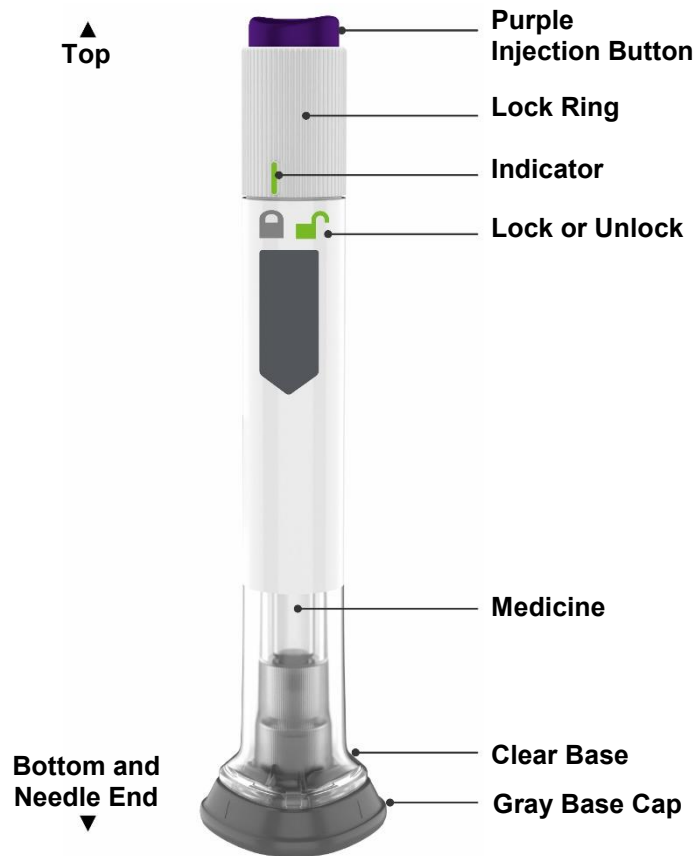

## Preparing to inject ZEPBOUND

**Remove the pen** from the refrigerator.

Leave the gray base cap on until you are ready to inject.

**Check the pen** label to make sure you have the right medicine and dose, and that it has not expired.



**Inspect the pen** to make sure that it is not damaged.

Make sure the medicine:


- is not frozen
- is colorless to slightly yellow
- is not cloudy
- does not have particles

**Wash your hands.**


**Step  
1**

**Choose your injection site**

Your healthcare provider can help you choose the injection site that is best for you.

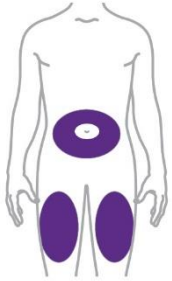

You or another person can inject the medicine in your stomach (abdomen) or thigh.

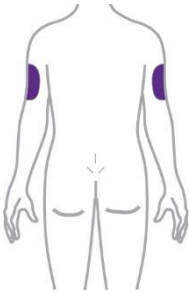

Another person should give you the injection in the back of your upper arm.

Change (rotate) your injection site each week.


You may use the same area of your body but be sure to choose a different injection site in that area.

**Step  
2**

**Pull off the gray base cap**

Make sure the pen is **locked**.

**Do not** unlock the pen until you place the clear base on your skin and are ready to inject.

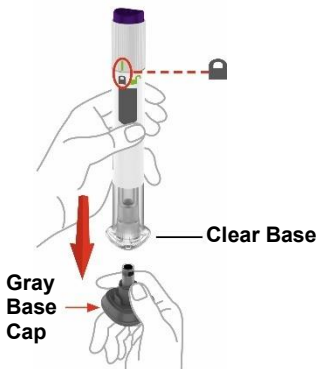

**Pull** the gray base cap straight off and throw it away in your household trash.


**Do not** put the gray base cap back on – this could damage the needle.


**Do not** touch the needle.

**Step  
3**

**Place clear base on skin, then unlock**

**Place** the clear base flat against your skin at the injection site.

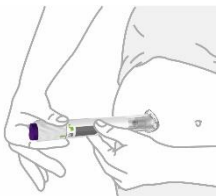


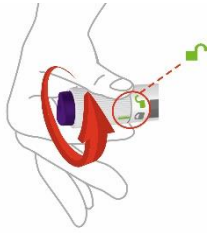

**Unlock** by turning the lock ring.

**Step  
4**

**Press and hold up to 10 seconds**

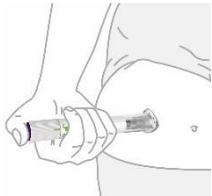

**Press and hold** the purple injection button for up to 10 seconds.

**Listen** for:

- First click = injection started
- Second click = injection completed



Gray  
Plunger

You will know your injection is complete when the gray plunger is visible.

**After your injection, place the used pen in a sharps container.**

See **Disposing of your used pen.**

**Disposing of your used pen**

- Put your used pen in an FDA-cleared sharps disposal container right away after use. **Do not** throw away (dispose of) pens in your household trash.
- If you do not have an FDA-cleared sharps disposal container, you may use a household container that is:
  - made of a heavy-duty plastic,
  - can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
  - upright and stable during use,
  - leak-resistant, and
  - properly labeled to warn of hazardous waste inside the container.
- When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles and syringes. For more information about safe sharps disposal, and for specific information about sharps disposal in the state that you live in, go to the FDA's website at: <http://www.fda.gov/safesharpsdisposal>.
- **Do not** recycle your used sharps disposal container.

## Storage and handling

- Store your pen in the refrigerator between 36°F to 46°F (2°C to 8°C).
- You may store your pen at room temperature up to 86°F (30°C) for up to 21 days. If you store the pen at room temperature, do not return the pen to the refrigerator.
- Discard the pen if not used within 21 days after removing from the refrigerator.
- **Do not** freeze your pen. If the pen has been frozen, throw the pen away and use a new pen.
- Store your pen in the original carton to protect your pen from light.
- The pen has glass parts. Handle it carefully. If you drop the pen on a hard surface, **do not** use it. Use a new pen for your injection.
- Keep your ZEPBOUND pen and all medicines out of the reach of children.

## Commonly asked questions

### What if I see air bubbles in my pen?

Air bubbles are normal.

### What if my pen is not at room temperature?

It is not necessary to warm the pen to room temperature.

### What if I unlock the pen and press the purple injection button before pulling off the gray base cap?

**Do not** remove the gray base cap. Throw away the pen and get a new pen.

### What if there is a drop of liquid on the tip of the needle when I remove the gray base cap?

A drop of liquid on the tip of the needle is normal. **Do not** touch the needle.

### Do I need to hold the injection button down until the injection is complete?

This is not necessary, but it may help you keep the pen steady against your skin.

### I heard more than 2 clicks during my injection—2 loud clicks and 1 soft one. Did I get my complete injection?

Some people may hear a soft click right before the second loud click. That is the normal operation of the pen. **Do not** remove the pen from your skin until you hear the second loud click.

### I am not sure if my pen worked the right way.



Check to see if you have received your dose. Your dose was delivered the right way if the gray plunger is visible. Also, see **Step 4** of the instructions.

If you do not see the gray plunger, contact Lilly at 1-800-Lilly-Rx (1-800-545-5979) for further instructions. Until then, store your pen safely to avoid an accidental needle stick.

### What if there is a drop of liquid or blood on my skin after my injection?

This is normal. Press a cotton ball or gauze over the injection site. **Do not** rub the injection site.

## Other information

- If you have vision problems, **do not** use your pen without help from a person trained to use the ZEPBOUND pen.

## Where to learn more

- If you have questions or problems with your ZEPBOUND pen, contact Lilly at 1-800-Lilly-Rx (1-800-545-5979) or call your healthcare provider.
- For more information about the ZEPBOUND pen, visit our website at [www.zepbound.com](http://www.zepbound.com).



Scan this code to launch  
[www.zepbound.com](http://www.zepbound.com)

Marketed by:  
Lilly USA, LLC  
Indianapolis, IN 46285, USA

ZEPBOUND is a trademark of Eli Lilly and Company.

Copyright © 2023, Eli Lilly and Company. All rights reserved.

This Instructions for Use has been approved by the U.S. Food and Drug Administration.

Revised: November 2023

ZEP-0002-PEN-IFU-20231109

